*The natural way of Sweden*

- **Gebrauchsanweisung und Pflegehinweise**
- **Instructions for use and care**
- **Mode d'emploi et conseils pour l'entretien**
- **Gebruiksaanwijzing en aanwijzingen voor onderhoud**
- **Instrucciones de uso y consejos de mantenimiento**
- **Istruzioni per l'uso e consigli per la manutenzione**
- **Bruksanvisning och skötselråd**
- **사용설명서 및 손질(세탁)방법**
- **Instrukcja użytkowania oraz środki ostrożności**
- **Brugs- og vedligeholdelsesvejledning**

# Die SISSEL® Nackenkissen

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# SISSEL® Orthopedic Pillows


**CONGRATULATIONS ON THE PURCHASE OF THIS QUALITY PRODUCT.  
COMPLIANCE WITH OUR CARE AND USE INSTRUCTIONS WILL GUARANTEE  
CONTINUED DAILY ENJOYMENT.**

**SISSEL® ORTHOPEDIC NECK PILLOWS MADE FROM HIGH QUALITY POLYURETHANE OR VISCO-ELASTIC FOAM.**


A deep, healthy sleep is one of the basic requirements for relaxation from the daily challenges. Pains coming from the neck, whose cause is often misdiagnosed, disturb this process. This is where the SISSEL® Neck pillow helps.

By supporting the lower cervical vertebrae and, thereby, relieving the neck muscles, the SISSEL® neck pillow provides substantial support to achieve a deep and relaxing sleep.

SISSEL® neck pillows contain no latex and come with a five year warranty on shape retention.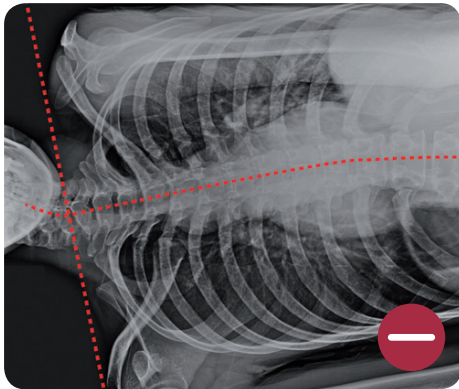

Ordinary pillows do not support the neck sufficiently and do not provide relief of the neck vertebrae.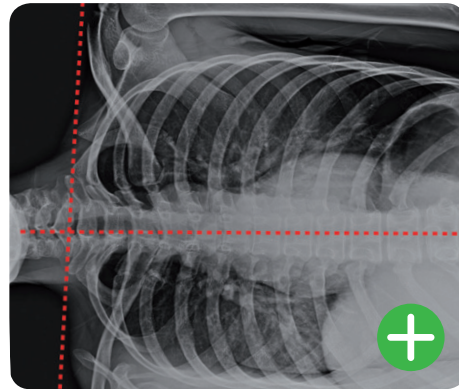


The SISSEL® neck pillow supports the neck perfectly, whether you sleep on the side or back.

## INDICATIONS

- | Pain in the cervical region (neck).
- | Tense neck muscles resulting in tension headaches.
- | Sleep problems caused by tensions.
- | Also in case of osteoarthritis and prolapse issues – after consultation with your physician.
- | Prevents a bending of your spine, when sleeping on your side.

## CONTRAINDICATIONS

Acute injuries/distortions or acute prolapse of the cervical spine, meningitis, after a stroke, acute nerve root inflammation in the neck area, spastic torticollis, neurological diseases or misalignments, tumors.

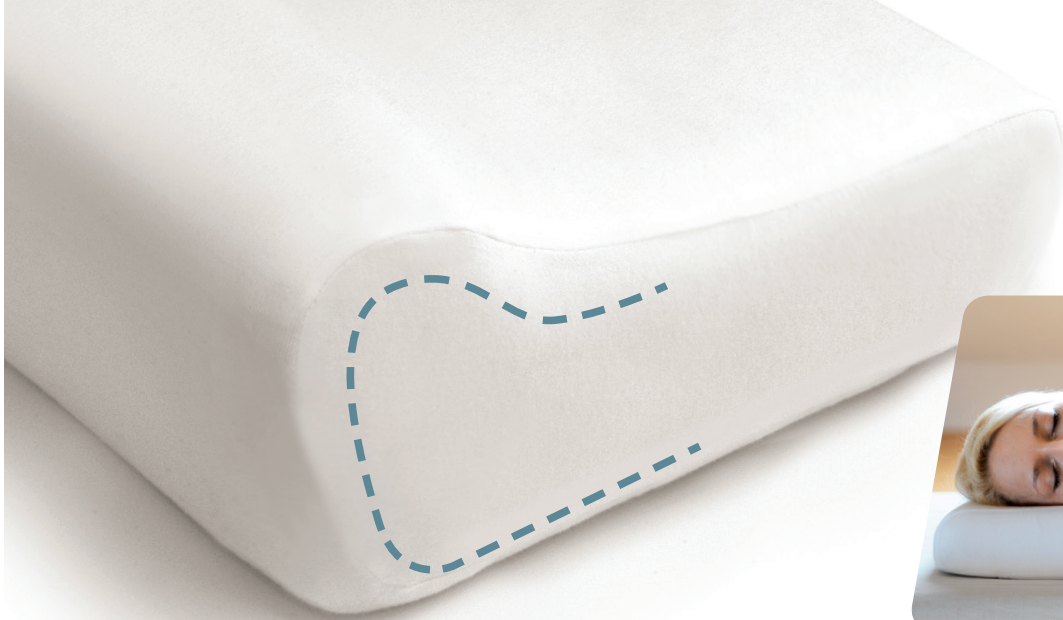### POSITIONING

Place the flat end of your SISSEL® orthopedic pillow under your head. The higher end should be at your shoulder. This ensures optimum support of your cervical spinal column.

### GETTING USED TO

It usually takes two to three nights to become accustomed to the right way of sleeping. Only very few cases require a little more time.

### OUR COVERS

Please note that all our products consist of high-quality materials, which require special care, due to their special qualities. Please remove the cover carefully and without force. We recommend to hold on to the entire pillow not just one end. The pillow's inner cover is made of organic cotton, pre-washed without the use of bleach. Any visible specs are residue of cotton plants and do not have any negative impact on pillow function or hygienics.

### CLEANING OF COVERS

For washing check label on cover.

### CLEANING OF PILLOW CORE

After washing the pillow core by hand with a mild detergent, carefully squeeze by hand, do not wring out. To dry, place pillow on a grate or similar (place a towel underneath) and give plenty of time to dry (about a week). Do not dry near a heat source or expose to direct sun light.

A slight discoloration of the pillow core is normal and has no impact on the special functions of the pillow.

### SCOPE OF DELIVERY

1 Orthopedic pillow, 1 cover, 1 manual, 1 transport box

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SISSEL® GmbH